



Goodrich Village Hall

'MobileConnect Hearing Assistance'
adjust film audio volumes to your own taste
on your phone or tablet device

STEP 1

Search & Download App
'Sennheiser MobileConnect'




STEP 2

Go to settings → Wi-Fi → Select MobileConnect




STEP 3

Select **DVD Player**
and then adjust volume to suit

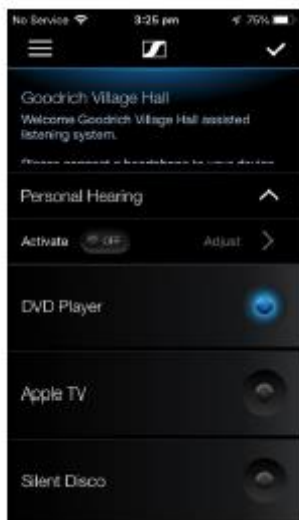

STEP 4

If the sound quality is not to your liking
change '**Personal Hearing**' to **ON** and select '**Adjust**'.
You can then move the sound button with your
finger within the screen space to adjust the sound

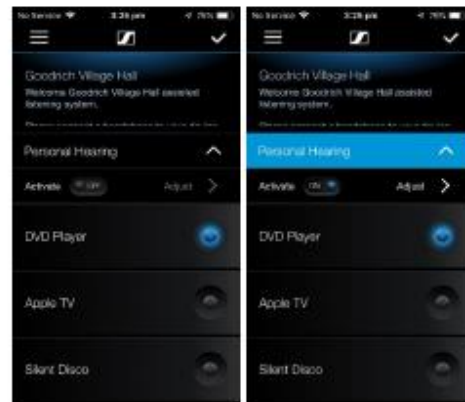

Move **sound button** with your finger

